

M15691	Wheeled	M01278	Divan Style
---------------	----------------	---------------	--------------------

General Safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury. Retain these instructions in a safe place for future reference.

1.0 Intended Use

Designed for use when sitting in either a chair or bed these tables provide an adjustable height flat surface for reading, writing, eating etc. The table top can be tilted for increased comfort during use.

2.0 Safety Precautions

Do not overload the table top, 5kg Max Load.

Check handwheels are secure at regular intervals.

Do not attempt to use the table as a walking aid.

Do not attempt to sit or stand on the table.

3.0 Unpacking

Remove packaging and check all parts carefully before assembling the product. Do not assemble if any parts are missing or damaged.

Contents:

- Table Leg (M15691 only with castors) X 2
- Cross Bar (with castors) X 1
- Table Top X 1
- Telescopic Tube X 2
- Hand Wheel – Long X 2
- Hand Wheel – Short X2

4.0 Assembly

4.1 Frame Assembly:

Position the cross brace over the two side frames as shown in figure 2.

Secure by inserting and tightening the two long hand wheels.



Figure 1 – In Use



Figure 2 – Assembling the cross bar

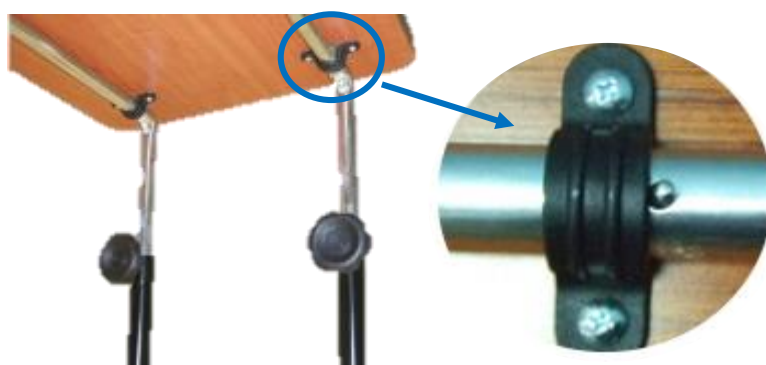


Figure 3 – Assembling the Table Top

4.2 Table Top Assembly:

Turn the table top upside down and lay flat.

Slide the two silver telescopic tubes through the brackets. *Lead with the end which has a single button clip.*

Depress the button clip to enable correct engagement through the brackets, figure 3. *Ensure it fully clicks into place on the far side of the second bracket.*

4.3 Connect the Table Top to the Base:

Insert the silver telescopic tubes into the open ends of the two side frames.

Push down gently on the table top.

Secure by inserting and tightening the two short hand wheels, figure 3.

5.0 Using the Table

5.1. To Adjust the Height

Loosen both of the handwheels at the top of the two black side frames by turning anti-clockwise.

Lift the table top up or push down gently to move it to the required height.

Tighten the two handwheels by turning clockwise to secure in position.

5.2 To Adjust the Angle

Please Note: *The table top can be tilted towards the user to a maximum of 45°.*

Loosen the handwheel at the top of the black side frame furthest away from the user.

Lift the far edge of the table top or push down gently to increase/decrease the angle of tilt.

5.3 Operating the Brakes

The 'rear' 2 castors are fitted with brakes.

To apply: firmly push the brake lever down until it clicks into position.

To release: lift the brake lever up until the castor is free to rotate.

6.0 Cleaning

Clean regularly using a general purpose, neutral detergent and warm water.

Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

Allow the product to dry thoroughly prior to use or storage.

Always check local authority guidelines for additional advice regards cleaning healthcare products.

7.0 Maintenance

Check castors are securely fitted and are free to rotate. Remove any build-up of dirt/debris.

Check the height and tilt adjustment operates correctly and the hand wheels are tight.

Inspect product regularly for signs of wear. Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

8.0 Product Disposal

Product should be thoroughly cleaned prior to disposal.

Product is manufactured from metal, plastic and wood and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

9.0 Product Guarantee

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship. This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS Healthcare

Sherwood House, Cartwright Way,
Forest Business park, Bardon Hill,
Coalville, Leicestershire, LE67 1UB, UK

Tel. Head Office: 0330 100 0098

Tel. Customer Services: 0345 121 8111

Email: customerservices@nrshealthcare.co.uk

NRS Healthcare is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:

www.nrshealthcare.co.uk

UI/M15691, M01278/06/18/Iss4